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Keep away from sunlight
Mantener alejado de la luz del sol

Rx Only

Caution: Federal (USA) Law restricts this device to sale by or on the order of a physician (or properly licensed healthcare practitioner)

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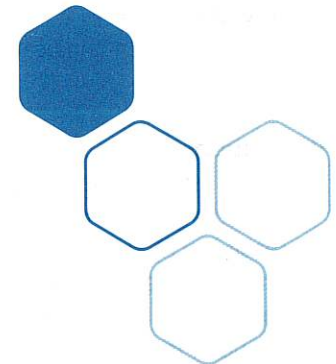
Antimicrobial bulky gauze bandages
Vendaje de gasa gruesa antimicrobiano

Product Description

Sterilux® AMD antimicrobial gauze bandages are sterile, single-use, wound bandages consisting of 100% cotton gauze impregnated with the mixture of polyhexamethylene biguanide hydrochloride (PHMB) and benzalkonium chloride (BKC). The bandages are packaged in paper/plastic trays and are available in both sponges and roll form. The antimicrobial activity of the PHMB and BKC helps resist bacterial colonization within the bandages.

Ingredients

0.1%PHMB + 0.02%BKC.

Indications

It is intended for use as primary or secondary bandages for exuding wounds, lacerations, abrasions, first and second degree burns, surgical incisions, wound packing, donor sites, catheter sites, I.V. sites and central lines. It can be used to secure and prevent movement of primary bandages as well.

Contraindications

The bandage is contraindicated for use on 3rd degree burns.

Precautions

- Single use and external use only.
- Should not be used on patients with known sensitivity to PHMB and BKC.
- The bandage only helps resist bacterial colonization within the bandage rather than intended treatment for clinical infection.
- Soak with sterile saline or water when the bandage is not easily removed.

Directions for use

1. Clean and prepare the wound by normal procedures.
2. Open the package and take out the sterile bandage.
- 3a. For use as primary bandage, fully cover or fill

the wound with appropriate bandage directly, fixed with tape, elastic wrap or film bandage.

- 3b. For use as second bandage, covered and then wrap with the sponge roll around primary bandages to secure and prevent it movement.
4. Change the bandage as required, at least every 24 hours. Change sooner if there is exudate strike-through. If healing does not progress, consult with your medical professional.

Antibacterial efficiency test

The product can achieve broad spectrum antimicrobial effect within the bandage for Gram+ and Gram- Bacteria and Fungi. It has been shown to have 4 log bacterial reduction in vitro against: Staphylococcus aureus, Escherichia coli, Candida albicans, Pseudomona aeruginosa and Bacillus subtilis, Streptococcus pyogenes, Serratia marcescens, Aspergillus niger. The effective inhibition of bacteria within the bandage is 7 days.

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