


Island Dressings

McKESSON


MFR # 16-89041, 16-89042, 16-89046, 16-89048, 16-89068

McKesson Island Dressings combine a non-adherent absorbent gauze pad and a non-woven adhesive border all in one. They can be used as either a primary or secondary dressing.


INSTRUCTIONS FOR USE

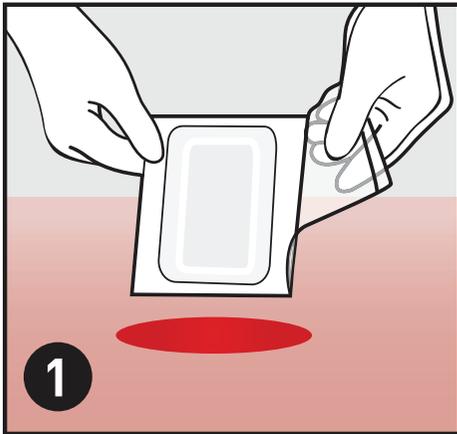


Cleanse and debride the wound area as necessary. Select an appropriately-sized dressing. Choose a dressing that will allow the wound contact area to completely cover the wound and extend onto healthy tissue by about 1" (25 mm).

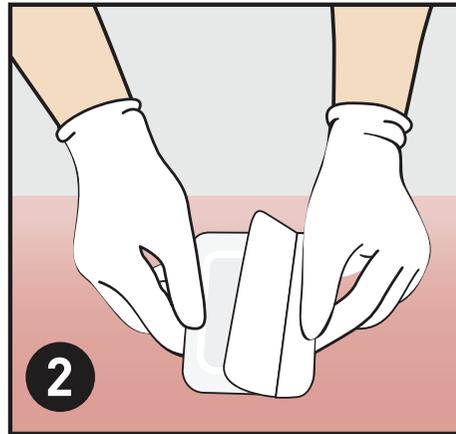

Protect skin as needed with liquid barrier film. To apply the dressing, first remove a portion of the white liner to expose the adhesive.

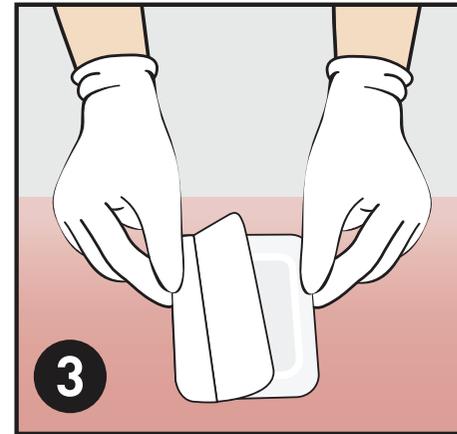

Position and smooth into place, without tension, while removing the remainder of the white liner.

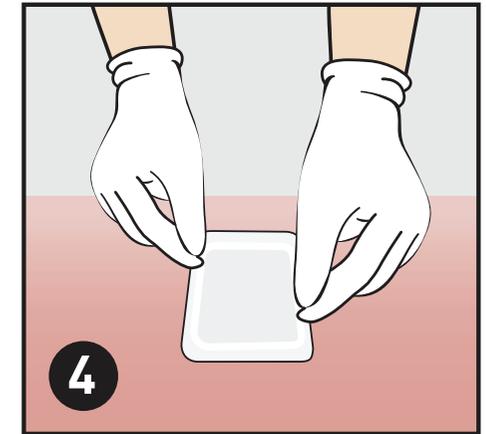

Carefully smooth around the edge of the dressing to ensure good contact between the adhesive border and the skin.