VITÁLE®

COLLAGEN POWDER DRESSINGS

INSTRUCTIONS FOR USE

PRODUCT DESCRIPTION

Vitále® Collagen Powder Dressings are absorbent wound fillers made of type I microfibrillar collagen for the management of minimally to heavily exuding acute and chronic wounds and control of minor bleeding. Vitále® Collagen Powder Dressing forms a gel on contact with exudate that provides a moist, physiologically favorable environment to support wound healing. Vitále® Collagen Powder Dressing is applied directly to the wound to act as a wound filler and to absorb exudate and exudate components that may impair wound healing.

Vitále® Collagen Dressings are sterilized by exposure to e beam radiation and must not be re-sterilized. The dressing is an off-white loose powder packaged in convenient single-use, sterile, peel open packets, containing 1 gram of powder. After use, discard open packaging and unused product per facility waste protocol.

INDICATIONS

Vitále® Collagen Powder Dressings are intended for the management of burns, sores, blisters, scrapes, ulcers, and other wounds including:

CAUTION

Vitále® Collagen Powder Dressing is intended for topical use only. Avoid inhalation and flush immediately with water if product comes in contact with eyes.

SIDE EFFECTS & INTERACTIONS

Vitále® Collagen Dressings should not be used on patients with known sensitivity or allergy to bovine collagen. Should not be used on third-degree burns or wounds with active infection.

STORAGE

Vitále® Collagen Dressings should be stored in a cool & dry area. The products are sterile unless the package is opened or damaged.

APPLICATION

1. Clean the wound with sterile physiological saline using existing clinical protocol. Dry the surrounding skin gently.
2. Peel open packet and apply sufficient powder to ensure approximately 1/4" depth of coverage of the wound bed. Do not overpack or compress powder as it may expand as it absorbs wound exudate.
3. Choose and apply appropriate secondary dressing or cover with an absorbent dressing to secure and protect collagen powder.
4. Change dressings daily, as directed, when fully saturated or according to facility change policy, but at least every 7 days.
5. When changing, remove Secondary Dressing, and gently remove gel formed of dressing and exudate. Clean the wound with sterile physiological saline using existing clinical protocol. Dry the surrounding skin gently.

Federal Law (USA) restricts the sale and use of this device at the direction of authorized physicians and licensed healthcare practitioners.

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