VITÁLE®

COLLAGEN DRESSINGS

INSTRUCTIONS FOR USE

PRODUCT DESCRIPTION

Vitále® Collagen Dressings are absorbent wound dressings made of type I 100% collagen intended for the management of minimally to heavily exuding acute and chronic wounds and control of minor bleeding. Vitále® Collagen Dressings form a gel on contact with exudate that provides a moist, physiologically favorable environment to support wound healing. Vitále® Collagen Dressings are applied directly to the wound to absorb exudate and exudate components that may impair wound healing, and to control minor bleeding.

Vitále® Collagen Dressings are sterilized by exposure to gamma radiation and must not be re-sterilized. The dressing is an off-white sheet, in various sizes, packaged in convenient single-use, sterile, peel open packets, containing 1 dressing. After use, discard open packaging and unused product per facility waste protocol.

INDICATIONS

Vitále® Collagen Dressings are indicated for the management of minimally to heavily exuding acute and chronic wounds including:

Superficial

- Pressure Sores
- Venous Stasis Ulcers
- Diabetic Ulcers

Minimally to Heavy Exuding

- Surgical, dehiscent surgical & traumatic wounds
- Partial thickness burns

Vitále® Collagen Dressings are also indicated for control of minor bleeding, including post-debridement.

CAUTION

For topical use only. Not for Ophthalmic use. Avoid eye contact. Do not use with uncontrolled clinical infection. Discontinue use and contact a physician if any adverse reactions and abnormal symptoms occur. The proposed maximum usage is up to 900 cm² per day per user.

An increase in drainage may be seen in the first several days of treatment. Some reactions (transitory pain, bleeding, blistering, swelling or redness) have been reported in isolated cases using a similar product. Discontinue use and contact a physician if any adverse reactions and abnormal symptoms occur.

SIDE EFFECTS & INTERACTIONS

Vitále® Collagen Dressings should not be used on patients with known sensitivity or allergy to bovine collagen. Should not be used on third-degree burns or wounds with active infection.

STORAGE

Vitále® Collagen Dressings should be stored in a cool & dry area. The products are sterile unless the package is opened or damaged.

APPLICATION

1. Clean the wound with sterile physiological saline using existing clinical protocol. Dry the surrounding skin gently
2. Apply the dressing to the entire wound surface. Cut the dressing to fit if necessary.
3. Choose and apply appropriate secondary dressing.
4. Change dressings daily, as directed, when fully saturated or according to facility change policy, collagen dressing is composed of 100% collagen and is biodegradable. Duration of treatment and reapplication is determined by the physician according to the wound conditions.
5. When changing, remove Secondary Dressing, and gently remove gel formed of dressing and exudate. Clean the wound with sterile physiological saline using existing clinical protocol. Dry the surrounding skin gently.

Federal Law (USA) restricts the sale and use of this device at the direction of authorized physicians and licensed healthcare practitioners.

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