

# APPLICATION INSTRUCTIONS

## WHAT IS INCLUDED

- Primary Dressing
- Secondary Dressing
- Conforming Stretch Bandage
- Waterproof Adhesive Tape
- Gloves

## WHAT YOU MAY NEED

- Scissors
- Hand Soap & Warm Water

### PREPARATION



1 Open your daily carePAC<sup>TM</sup>, remove all components, and set them on a clean surface.



2 Wash hands with soap and water 20 seconds<sup>1</sup>, rinse, then dry with clean towel or air.



3 Put on supplied latex free nitrile gloves to reduce the risk of infection.



4 Clean wound according to your healthcare provider's instructions before applying primary dressing.

Please follow the below application instructions for Collagen Powder or Collagen Matrix based upon your specific order.

### COLLAGEN POWDER



5 Carefully open collagen powder package without compromising sterility.



6 Apply collagen powder from the opened packet to reach roughly ¼-inch depth across the wound bed. Do not tightly pack or press into place<sup>2</sup>.



7 Cover the coated wound entirely with the secondary dressing<sup>2</sup>.




8 Secure the secondary dressing with the conforming stretch bandage and tape as needed.


### COLLAGEN MATRIX




5 Open the collagen matrix package without compromising sterility.

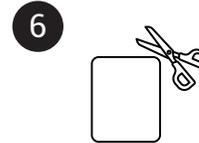

6 The collagen matrix may be trimmed to size if instructed by your healthcare provider<sup>3</sup>.

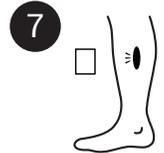

7 Apply the collagen matrix to the entire wound surface<sup>3</sup>.

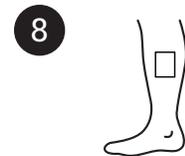

8 Cover the matrix entirely with the secondary dressing<sup>3</sup>.



9 Secure the secondary dressing with the conforming stretch bandage and tape as needed.

Please refer to each product's Instructions for Use for complete indications and application guidance.

**Disclaimer:** None of the statements on this page are intended to make any claims that studies have been conducted on these products.

### REFERENCES

1. Centers for Disease Control and Prevention. (2024, February 16). About Handwashing. Clean Hands. <https://www.cdc.gov/clean-hands/about/index.html>
2. CellEra, LLC. (2022). Vitale Collagen Powder Dressings: Instructions for Use (English, 10-20-22). <https://cellera.info/wp-content/uploads/Vitale-Collagen-Powder-Dressings-IFU-English-10-20-22.pdf>
3. CellEra, LLC. (2021). Vitale Collagen Sheet Dressings: Instructions for Use (English, 3-4-21). <https://cellera.info/wp-content/uploads/Vitale-Collagen-Sheet-Dressings-IFU-English-3-4-21.pdf>



Tiger Wound Care Medical, LLC  
555 E North Lane, Ste 5000, Bldg D • Conshohocken, PA 19428  
1.833.398.4437 • [www.tigerwoundcare.com](http://www.tigerwoundcare.com)



SCAN FOR MORE INFORMATION