

APPLICATION INSTRUCTIONS

WHAT IS INCLUDED

- Primary Dressing
- Secondary Dressing

WHAT YOU MAY NEED

- Scissors
- Hand Soap & Warm Water

PREPARATION



- 1
Open your HealPACK box, remove a primary and secondary dressing, and set them on a clean surface.



- 2
Wash hands with soap and water 20 seconds¹, rinse, then dry with clean towel or air.



- 3
Clean wound according to your healthcare provider's instructions before applying primary dressing.

Please follow the below application instructions for Collagen Powder or Collagen Matrix based upon your specific order.

COLLAGEN POWDER



4
Carefully open collagen powder package without compromising sterility.



5
Apply collagen powder from the opened packet to reach roughly ¼-inch depth across the wound bed. Do not tightly pack or press into place².




6
Cover the coated wound entirely with the secondary dressing².


COLLAGEN MATRIX




4
Open the collagen matrix package without compromising sterility.

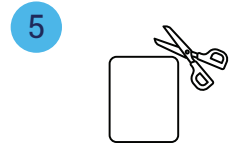

5
The collagen matrix may be trimmed to size if instructed by your healthcare provider³.

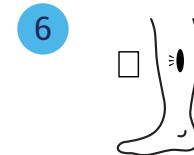

6
Apply the collagen matrix to the entire wound surface³.

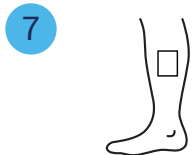

7
Cover the matrix entirely with the secondary dressing³.

Please refer to each product's Instructions for Use for complete indications and application guidance.

Disclaimer: None of the statements on this page are intended to make any claims that studies have been conducted on these products.

REFERENCES

- Centers for Disease Control and Prevention. (2024, February 16). About Handwashing. Clean Hands. <https://www.cdc.gov/clean-hands/about/index.html>
- CellEra, LLC. (2022). Vitale Collagen Powder Dressings: Instructions for Use (English, 10-20-22). <https://cellera.info/wp-content/uploads/Vitale-Collagen-Powder-Dressings-IFU-English-10-20-22.pdf>
- CellEra, LLC. (2021). Vitale Collagen Sheet Dressings: Instructions for Use (English, 3-4-21). <https://cellera.info/wp-content/uploads/Vitale-Collagen-Sheet-Dressings-IFU-English-3-4-21.pdf>

Questions about your HealPACK dressings?

- ☎ (877) 219-6016
- ✉ healpack@tigerbios.com
- 🌐 www.tigerbiosciences.com

Encore Surgical Dressings

555 E. North Lane, Ste 5000, Bldg D, | Conshohocken, PA 19428



MDC-1585, Revision 00