

– Instructions for Use –

Product Description: Vitále® Collagen Dressings are absorbent wound dressings made of type 1 microfibrillar collagen intended for the management of minimally to heavily exuding acute and chronic wounds and control of minor bleeding. Vitále Collagen Dressings form a gel on contact with exudate that provides a moist, physiologically favorable environment to support wound healing. Vitále Collagen Dressings are applied directly to the wound to absorb exudate and exudate components that may impair wound healing, and to work as a hemostatic agent to control minor bleeding.

Vitále Collagen Dressings are sterilized by exposure to gamma radiation and must not be re-sterilized. The dressing is an off-white sheet, in various sizes, packaged in convenient single-use, sterile, peel open packets, containing 1 dressing. After use, discard open packaging and unused product per facility waste protocol.

Indications: Vitále Collagen Dressings are indicated for the management of minimally to heavily exuding acute and chronic wounds, including:



Superficial or Deep

- Pressure Ulcers
- Venous Stasis Ulcers
- Diabetic Ulcers



Minimally to Heavy Exuding

- Surgical, dehisced surgical, and traumatic wounds
- Partial thickness burns.

Vitále Collagen Dressings are also indicated for control of minor bleeding, including post-debridement.

Precautions: For topical use only. Not for Ophthalmic use. Avoid eye contact.

Contraindications: Vitále Collagen Dressings should not be used on patients with known sensitivity or allergy to bovine collagen.

Application:

1. Cleanse the wound following facility protocol to prepare the wound bed, using saline or a wound cleanser such as Vitále Wound Cleanser Spray. Deep wounds should be well irrigated. Use a protectant film such as Vitále Skin Protectant Wipes on intact skin surrounding the wound.
2. Apply the Vitále Collagen Dressing to the entire wound surface. Cut the dressing to fit if necessary.
3. Choose and apply appropriate secondary dressing, such as Vitále Cover or Composite Dressings, Vitále Hydrocolloid Dressings, etc.
4. Change dressings daily, as directed, when fully saturated or according to facility change policy, at maximum of every 7 days.
5. When changing, remove Secondary Dressing, and gently remove gel formed of dressing and exudate. Cleanse with saline irrigation or a wound cleanser such as Vitále Wound Cleanser Spray. Reapply & cover as above.

Warnings: An increase in drainage may be seen in the first several days of treatment. Some reactions (transitory pain, bleeding, blistering, swelling or redness) have been reported in isolated cases using a similar product. If these conditions continue for more than 3-5 days, discontinue use of the product and notify the physician.

Federal Law (USA) restricts the sale and use of this device at the direction of authorized physicians and licensed healthcare practitioners.

– Product Line –


Catalog Number:	Description:	Size:	Dressings/Box:
22011	Vitále Collagen Dressings	1" x 1" (2.5 x 2.5 cm)	10
22022	Vitále Collagen Dressings	2" x 2" (5 x 5 cm)	10
22044	Vitále Collagen Dressings	4" x 4" (10 x 10 cm)	10
22077	Vitále Collagen Dressings	7" x 7" (18 x 18 cm)	10

Made in USA for



CellEra, LLC Cincinnati, OH
www.celleralc.com

STERILE Sterile

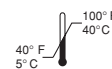

Storage Temperature

⊗ Single-Use

☂ Keep Dry

R_x Rx Only

☀ Keep From Heat